


NEEDLE FREE VENTED VIAL ADAPTER DIRECTIONS FOR USE



NO NEEDLES

To draw out of the vial you **MUST** take the needle off the syringe
The Clave adapter will not accept needles and needle sticks can result in leaking

In the event that you draw up more drug than needed you **MUST** invert the vial to its right side up position to infuse the unwanted drug back into the vial

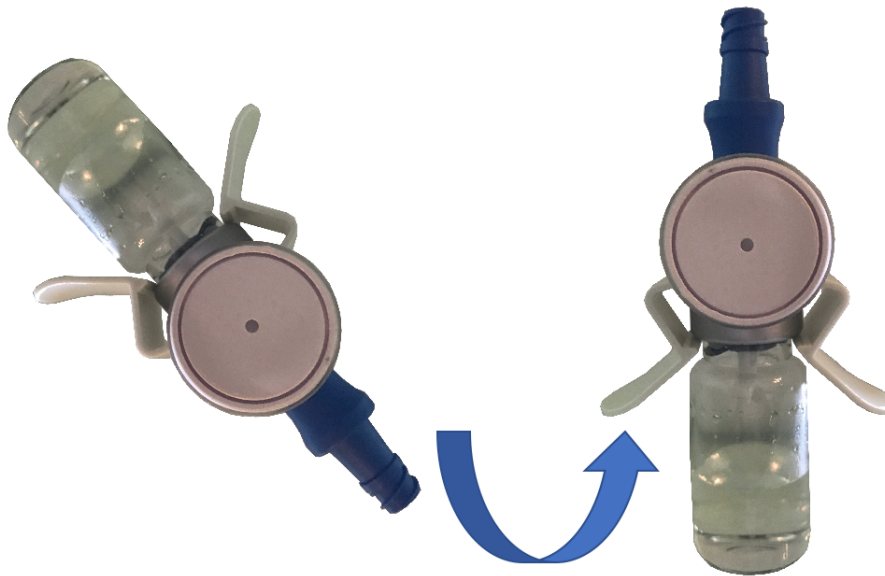

If you do infuse the vial while upside-down, this will block the air filter. To ensure the vial adapter will function as intended after blocking the filter, do the following:

- Draw out of the vial until you start to see bubbles come out of the spike
- Invert vial back right side up and infuse drug back into vial

Now you can draw up your drug and the vial should equalize properly

Depending on volume in vial, you may have to repeat those steps a second time

PractiVet Item #AH70 HSAH Item #055114